



DirectHit™ Test Panel for Breast Cancer Drug Response Indicator Test

Specimen Collection Kit Instructions

Kit Components

- 7 slide mailers, blue
- Tamperproof security tie
- Breast Cancer Specimen Collection Kit Instructions
- Padded slide shipping envelope
- Blank DirectHit Test Panel for Breast Cancer requisition form
- FedEx padded pack (11¾" x 15¼") with pre-paid address label

Instructions

- Complete the DirectHit Test Panel for Breast Cancer requisition form.
- Each order for the standard DirectHit Test Panel for Breast Cancer requires six (6) slides.
- Ensure that each slide is labeled with the patient name and/or ID.
- Place a single slide in each slide mailer.
- Prepare a locked stack of slide mailers by placing the slide mailers on top of each other, then securing each slide mailer in its place by sliding it forward until a click is heard.
- Insert the seal end of the tamperproof security tie into the provided hole on the slide mailer assembly.
- Insert the tail end of the tamperproof security tie through the locking mechanism.
- Hold the locking head and pull the tail tight.
- Place the slide mailer assembly in the padded shipping envelope.
- Insert the padded shipping envelope and the completed DirectHit Test Panel for Breast Cancer requisition form into the FedEx padded pack (11¾" x 15¼") with pre-paid address label. For reference, the shipping address is listed below:

CCC Diagnostics, LLC
3918 Vero Rd, Suite B
Baltimore, MD 21227

- If there are any questions or problems, please contact :

Scott Deamond
410-633-4885 x1071
Scott@cccdiag.com